REF 3960 Rx Only

en

VORTTRAN APM-Plus

Airway Pressure Monitor

Intended Use

The VORTTRAN APM-Plus is used to measure airway pressure for any patient utilizing positive pressure devices where monitoring is desired.

Model No. 3960

Package Contents: 1 unit

For Multiple Patient Use

Contraindications

None

Indications For Use

The VORTTRAN APM-Plus is a multiple-patient-use device intended to monitor a positive-pressure device that connects to an adult or pediatric patient ≥ 10 kg. The device alarms when the conditions (pressure and/or rate, as applicable) are outside of the user selected high and low alarm limits. In PAP Mode, it displays only real-time breathing circuit pressure; in Resuscitator Mode, it displays ventilation parameters in addition to the pressure information. It may be used with positive pressure devices which do not include pressure measurement capabilities; e.g. resuscitation bags, automatic resuscitators, and CPAP, or as an independent backup pressure monitor for devices with pressure measurement capability. The device is designed to be portable and can be used in stationary or portable situations in hospitals and sub-acute institutions. For professional use only.

Essential Performance

Essential Performance is performance in which any loss or degradation would result in an unacceptable risk, and includes the following:

- Indication of measured values displayed on the LCD
 - PAP Mode: Real-Time Pressure
 - Resuscitator Mode: PIP, PEEP, Rate, Inspiratory Time, Expiratory Time, I:E Ratio
- Audible and visual alarms indicating breathing circuit pressure (or other parameters) that is outside of user set limits
- Operation within the measurement accuracies specified in the "VORTTRAN APM-Plus Specifications" section of this document.

The information in these instructions have been carefully checked and are believed to be accurate. In the interest of continued product development, VORTTRAN Medical Technology reserves the right to make changes and improvements to this product and instructions at any time, without notice or obligation.

Not made with natural rubber latex

Consult Instructions for Use - Caution - Catalogue Number - Non-Sterile - Prescription Only - MR Unsafe

Manufactured and Distributed by:
VORTTRAN Medical Technology 1, Inc.

21 Goldenland Court, #100 - Sacramento, CA 95834-2426
Tel: +1 (916) 648-8460 Fax: +1 (916) 243-1338
www.vortran.com

Table of Contents

Intended Use	1
Contraindications	1
Indications For Use	1
Essential Performance	1
Brief Device Description of the VORTTRAN APM-Plus	2
Table of Contents	2
Quick Setup Guide	3
VORTTRAN APM-Plus Operating Instructions	4-9
Components	4
Symbols on Label	4
Connecting the VORTTRAN APM-Plus to a Positive-Pressure Device	5
General Operation and Mode Selection	5-6
Resuscitator Mode Operation	7
PAP Mode Operation	8-9
Maintenance and Cleaning	9-10
Warnings	10
Precautions	11
Specifications	11
Troubleshooting	12
Mounting and Placement	12
Disposal	12
Warranty	12
Electromagnetic Compatibility	13-15

Brief Device Description of the VORTTRAN APM-Plus

The VORTTRAN APM-Plus is a battery (2 x AA) operated, portable, self-contained device that connects to a positive airway-pressure device (such as a resuscitator or CPAP) via pressure tubing. It is designed to measure and monitor pressure characteristics. During an alarm condition, the VORTTRAN APM-Plus has a flashing red LED and an audible sound, while simultaneously displaying the alarm condition on the LCD.

MONITORING/ALARMS, RESUSCITATOR MODE: The VORTTRAN APM-Plus will display PIP, PEEP, Respiratory Rate, Inspiratory and Expiratory Time, and I:E Ratio. When a connected pressure-cycled device stops cycling for a user-set time or the pressure falls under the user-set limit, the VORTTRAN APM-Plus will activate the NON-CYCLING ALARM or LOW PEEP ALARM, respectively. In addition, when respiratory rate or the pressure exceeds the user-set limit, the HIGH RATE or HIGH PIP alarm will activate, respectively.

MONITORING/ALARMS, PAP MODE: The VORTTRAN APM-Plus will display real-time pressure. When a connected constant-pressure device falls outside of the user-set range, the HIGH PRESSURE or LOW PRESSURE ALARM will activate.

EMC Declaration (*continued*)

Recommended separation distances between portable and mobile RF communications equipment and the VORTTRAN APM-Plus

The VORTTRAN APM-Plus is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the VORTTRAN APM-Plus can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the VORTTRAN APM-Plus as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (Watts)	Separation distance according to the frequency of transmitter (meters)		
	150 kHz to 80 MHz $d=1.2\sqrt{P}$	80 MHz to 800 MHz $d=1.2\sqrt{P}$	800 MHz to 2.5 GHz $d=2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.37	0.37	0.74
1	1.2	1.2	2.3
10	3.7	3.7	7.4
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum power rating of the transmitter in Watts (W) according to the transmitter manufacturer.

The VORTTRAN APM-Plus must be kept away from any RFID source, as it may interfere with the device.

Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

EMC Declaration (continued)

Guidance and Manufacturer's Declaration Electromagnetic Immunity

The VORTRAN APM-Plus is intended for use in the electromagnetic environment specified below. The customer or user of the VORTRAN APM-Plus should assure that it is used in such an environment.

Immunity Test	IEC 60601-2 Test Level	Compliance Level	Electromagnetic Environment Guidance Level
Radiated RF EM Fields - IEC 61000-4-3	80 MHz - 2.7 GHz @ 3 V/m	80 MHz - 2.7 GHz @ 3 V/m	<p>Portable and mobile RF communications equipment should be used no closer to any part of the monitor, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance: $d=1.2\sqrt{P}$ 80 MHz to 800 MHz $d=2.3\sqrt{P}$ 800 MHz to 2.5 GHz</p> <p>Where P is the maximum power rating of the transmitter manufacturer and d is the recommended separation distance in meters (m).</p> <p>Field strength from fixed RF transmitters, as determined by an electromagnetic site survey^a, should be less than the compliance level in each frequency range.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.


a) Field strengths from transmitters, such as base stations for radio (cellular, cordless) telephones and land mobile radios. Amateur radio, AM, and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the VORTRAN APM-Plus is used exceeds the applicable RF compliance level above, the equipment should be observed to verify normal operation. If abnormal operation is observed, additional measures may be necessary, such as reorienting or relocating the VORTRAN APM-Plus.

b) Over the frequency range of 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Quick Setup Guide

- Refer to the Component list on Page 4 to match the numbers shown in parenthesis in this guide with the corresponding component of the APM-Plus.
- Setup the device that is to be monitored per the manufacturer's instructions and connect the APM-Plus to the desired point in the circuit.
- Press the Power Button (2) to turn device on.
- Use Up (3) and Down (4) Buttons to choose Resuscitator or PAP mode. Press Enter (6) to make a selection.
- The Measurement Display screen on the LCD will show the values used to monitor the connected device. The last line shows the current/default Alarm Settings. Press and hold the Menu Button (6) to enter the Alarm Settings Menu to change from the default settings.
- Use Up (3) and Down (4) Buttons to choose the Alarm Setting that is to be changed. Press Enter (6) to make a selection. Use Up (3) and Down (4) Buttons to increase or decrease the setting to the desired value. Confirm the selection by pressing Enter (6). When finished, use the Up (3) or Down (4) Button to cycle through the Alarm Settings Menu to "EXIT" and Press Enter (6).
- Alarm conditions are shown on the Measurement Display screen when triggered. Press the Mute Button (6) quickly to temporarily mute the audible alarm.
- PAP Mode Only:** In the Measurement Display screen, the Real-Time Pressure will be shown. Press and hold the Auto-Set Button (3) to automatically set the Nominal Pressure Setting to the current Real-Time Pressure. An alert tone will be heard when successful. When the Real-Time Pressure deviates by at least $\pm 2 \text{ cm-H}_2\text{O}$ from the Nominal Pressure Setting, an alert tone will sound and an on-screen indication will be shown on the LCD.
- Press and hold the Power Button (2) for > 2 seconds to turn the device off.

VORTTRAN APM-Plus Operating Instructions

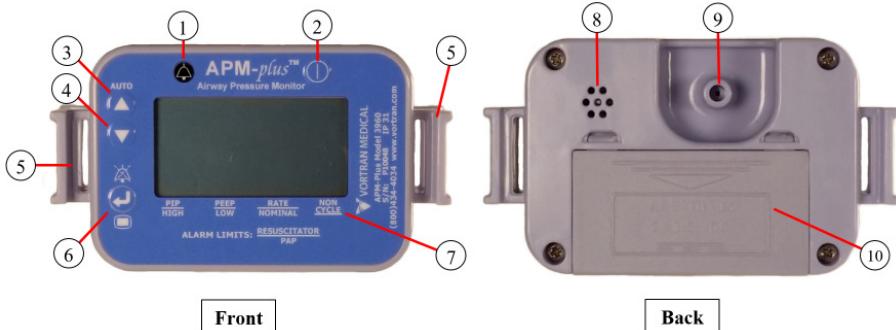Components:

- Alarm Indicator** – Red LED that indicates that an alarm has been triggered.
- Power Button** – Quick press to turn device on, hold for >2 seconds to turn device off.
- Up/Auto-Set Button** – Use when navigating menus and increasing Alarm Setting values. In PAP Mode only, press and hold for >2 seconds in the Measurement Display screen to automatically set the Nominal Pressure Setting to the current Real-Time Pressure.
- Down Button** – Use when navigating menus and decreasing Alarm Settings values.
- Mounting Clips** – Use to secure device to an equipment pole or bed frame using a pole clamp or velcro strap. Device can otherwise sit on any flat surface.
- Enter/Menu/Mute Button** – Use to make selections in menus, enter the Alarm Settings Menu, or press during an alarm condition to temporarily mute the alarm.
- Alarm Settings Label** – Label for the Alarm Settings values shown on the bottom of the LCD in the Measurement Display screen.
- Speaker** – Audible alarm is heard during alarm conditions and alerts.
- Luer** – Female Luer port for connecting to device that is to be monitored.
- Battery Door** – Remove battery door to replace batteries (2 x AA) when “LOW BATT” alarm is shown on-screen.

Symbols on Front Label:

EMC Declaration

Guidance and Manufacturer's Declaration Electromagnetic Emissions

The VORTTRAN APM-Plus is intended for use in the electromagnetic environment specified below. The customer or user of the VORTTRAN APM-Plus should assure that it is used in such an environment.

Emissions Test	Compliance Level	Electromagnetic Environment Guidance Level
Conducted / Radiated RF Emissions - CISPR 11	Class B 30MHz - 1GHz	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.

Guidance and Manufacturer's Declaration Electromagnetic Immunity

The VORTTRAN APM-Plus is intended for use in the electromagnetic environment specified below. The customer or user of the VORTTRAN APM-Plus should assure that it is used in such an environment.

Immunity Test	IEC 60601-2 Test Level	Compliance Level	Electromagnetic Environment Guidance Level
Electrostatic Discharge (ESD) - IEC 61000-4-2	± 8kV contact ±2kV, ±4kV, ±8kV, ±15kV air	± 8kV contact ±2kV, ±4kV, ±8kV, ±15kV air	Floors can be wood, concrete, ceramic, or synthetic. Relative humidity should be at least 30%.
Rated Power Frequency Magnetic Fields - IEC 61000-4-8	50 or 60 Hz @ 30 A/m	50 Hz @ 30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Troubleshooting

Problem	Potential Cause	Resolution
Device does not turn on	Batteries are missing or depleted	Place new AA batteries into device
Device gives zero for all readings	Pressure tubing is loose or disconnected	Reconnect or secure pressure tubing
Inaccurate readings	Pressure tubing is loose	Secure pressure tubing
No sound generated	Mute button inadvertently pressed	Wait for alarm to resume. See "Operating Instructions," Section 2 e.
LED display is missing segments	Display damage or failure	Contact manufacturer

Mounting and Placement

Device can be placed on any flat surface or mounted as shown below. It should be positioned with the LCD facing the operator at a height within the operator's line of sight.

Warranty

VORTTRAN Medical Technology 1, Inc. warrants that each new product or parts thereof will be free of defects in workmanship or materials. Should any failure to conform to this warranty appear with substantiation that the product has been stored, installed, maintained and operated in accordance with manufacturer's instructions and standard industry practice, and that no modifications, substitutions or alterations have been made to the product, VORTTRAN Medical shall correct such defect by suitable repair or replacement. Any misuse of the products or parts thereof may void the warranty. All warranty decisions are made by the manufacturer.


Disposal

After the warranty period, if the VORTTRAN APM-Plus fails to power on after replacing with new batteries or if the enclosure is broken, discard the device. At the end of the product life, we recommend that you dispose of your equipment in an environmentally sound manner. If you wish to discard this product, please contact your local authorities and ask for the correct method of disposal.

VORTTRAN APM-Plus Operating Instructions (continued)

1. Connecting the VORTTRAN APM-Plus to a Positive-Pressure Device

- The Connection Kit for Resuscitators is pre-assembled and consists of pressure tubing, Luer connector, hydrophobic filter, and tee adapter. See table below (Order No. VCK-3942).
- Connect the male Luer end of filter to female Luer (9) on the APM-Plus. The other end of the tubing connects to the section of the breathing circuit where monitoring is desired. Refer to "Mounting and Placement" (Page 12) for instructions on securing the device.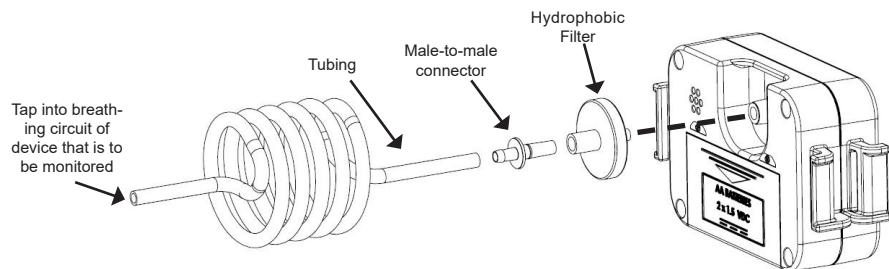

Included with Device Packaged Separately for Single-Use.			
Order No.	Connection to APM-Plus	Length of Pressure Tubing	Circuit Connection: Tee
VCK-3942	Filter	4'	

2. General Operation and Mode Selection

- Turn the VORTTRAN APM-Plus on by pressing and holding (less than a half second) the Power Button (2) on the front panel. The backlit LCD screen will power on with a half second beep sounding from the speaker. Simultaneously, a red LED lights up indicating the system is on. The VORTTRAN logo is shown on the LCD screen as the system is loading.
- After loading, the Mode Selection screen will be shown on the LCD. Use the Up (3) and Down (4) Buttons to navigate to the desired mode. For monitoring pressure-cycled devices, select "RESUSCITATOR" mode and press the Enter Button (6). See Section 3 of this guide for Resuscitator Mode instructions. For monitoring constant-pressure devices, select "PAP" mode and press the Enter Button (6). See Section 4 of this guide for PAP mode instructions.


Note: Device will shut off automatically to save battery life if left in the Mode Selection screen untouched for one minute.

VORTRAN APM-Plus Operating Instructions (continued)

- c. Once a mode is selected, the Measurement Display screen will show the measured values used to monitor the connected device. Any alarm condition triggered during the first two minutes of operation will be muted to allow the clinician to set up the device free of distraction. The last line on the LCD shows the current Alarm Settings, which will initially be the factory set default settings. Press and hold the Menu Button (6) for two seconds to enter the Alarm Settings menu to adjust from the default settings, if desired. For PAP Mode, it is ideal to set the Nominal Pressure Setting to the current or desired operating pressure of the connected device. See Section 4a and 4d.
- d. Use the Up (3) and Down (4) Buttons to choose the Alarm Setting that is to be changed. Press Enter (6) to make a selection. Use Up (3) and Down (4) Buttons to increase or decrease the setting to the desired value. Confirm the selection by pressing Enter (6). When finished, use the Up (3) or Down (4) Button to cycle through the Alarm Settings until "EXIT" is displayed, and press Enter (6) to return to the Measurement Display screen.
- e. Alarm conditions are shown on the Measurement Display screen when triggered. Press the Mute Button (6) quickly to temporarily mute the audible alarm. The symbol on the top-left of the LCD in the Measurement Display screen will show if the alarm is currently muted (see below). Temporary mute durations are as follows:
 - Resuscitator Mode (all alarms) - 30 seconds
 - PAP Mode (Low Pressure alarm) - 2 minutes
 - PAP Mode (High Pressure alarm) - 30 seconds

Normal Operation (alarm will be audible)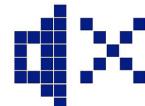

Temporarily Muted

- f. Turn off the VORTRAN APM-Plus by pressing and holding (for 2 seconds) the Power Button (2). The LCD will display "POWERING DOWN" with the red LED lit. When the LCD and red LED turn off, the device is powered off.

3. Resuscitator Mode Operation

- a. **Monitoring:** The Measurement Display screen in Resuscitator Mode cycles between the two screens every three seconds, each having three lines showing each measurement.

Precautions

- 1. Please review and follow the instructions and observe the warnings before using the APM-Plus. If the use or operation of the VORTRAN APM-Plus is unclear, contact your distributor or dealer for clarification.
- 2. Alarm Settings should be set to reasonable limits. Extreme values may prevent the APM-Plus from alarming when necessary.

VORTRAN APM-Plus Specifications

MONITORING - PAP MODE	RANGE	RESOLUTION	ACCURACY
Real-Time Pressure	0 to 55 cm-H ₂ O	0.1 cm-H ₂ O	± 1 cm-H ₂ O
MONITORING - RESUSCITATOR MODE	RANGE	RESOLUTION	ACCURACY
PIP and PEEP Airway Pressure	0 to 55 cm-H ₂ O	1 cm-H ₂ O	± 1 cm-H ₂ O
Respiratory Rate (Breaths Per Minute)	0 to 99 BPM	1 BPM	± 10%
I-Time & E-Time	0 to 9.9 second	0.1 second	± 10%
I:E Ratio	1:0.0 to 1:9.9	0.1	± 10%
HIGH PRIORITY ALARMS - PAP MODE	PRESET LIMIT	LCD SHOWS	ADJUSTABLE RANGE
High Pressure Alarm	12 cm-H ₂ O	"HIGH PRESSURE"	1 to 50 cm-H ₂ O
Low Pressure Alarm	2 cm-H ₂ O	"LOW PRESSURE"	1 to 15 cm-H ₂ O
Low Battery Alarm	N/A	"LOW BATT"	N/A
HIGH PRIORITY ALARMS - RESUSCITATOR MODE	PRESET LIMIT	LCD SHOWS	ADJUSTABLE RANGE
Non-Cycling Alarm	10 seconds	"NON-CYCLING"	1 to 20 s
High PIP Alarm	50 cm-H ₂ O	"HIGH PIP"	1 to 50 cm-H ₂ O
Low PEEP Alarm	2 cm-H ₂ O	"LOW PEEP"	1 to 15 cm-H ₂ O
High Respiratory Rate Alarm	35 BPM	"HIGH RATE"	12 to 99 BPM
Low Battery Alarm	N/A	"LOW BATT"	N/A
TEMPERATURE	HUMIDITY	AMBIENT PRESSURE	
Operating/Storage Environmental Limits	Operating: 5 to 40 °C Storage: -25 to 70 °C	15 to 90%	700 to 1060 hPa
Power source	2 x AA, IEC Type LR6 or ANSI type 15A (1.5V alkaline type batteries)		
External dimensions	4.3" W x 2.5" H x 1.3" D (11 cm W x 6.4 cm H x 3.3 cm D)		
Weight	150 g (with AA alkaline type batteries)		
Enclosure Degree of Protection	IP31		
Audible level	75 dB when measured at 1 meter distance		
Standards	IEC 60601-1, IEC 60601-1-2, and IEC 60601-1-8		

Maintenance and Cleaning (continued)


3. Verify the device for proper function after cleaning / before use.
 - a. Visually inspect VORTRAN APM-Plus to verify that housing cracks are not present. If cracks are present, do not use the device.
 - b. Visually inspect VORTRAN APM-Plus to verify that the LCD display is readable. If the LCD display is not readable, do not use the device.
 - c. Turn on VORTRAN APM-Plus and verify that the LED lights and the LCD is as below. An audible beep from the speaker should be heard during startup. If the LCD does not display as indicated or if "LOW BATT" is shown, do not use the device.

NOTE: If the battery level indicator in the top right of the LCD is not fully shaded, replace the batteries before using the device.

LCD: Screen 1

LCD: Screen 2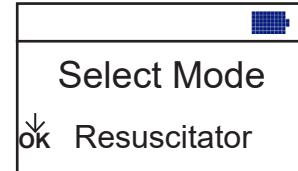

- d. During operation: Verify the battery charge. Check the battery level indicator in the top right of the LCD. Replace batteries if "LOW BATT" is displayed on the LCD at any time. Typical operation life for continuous operation is 7 days. Life will be diminished by frequent alarm conditions.

Battery Level Indicator: Full

Battery Level Indicator:
Extremely Low – Replace Batteries

Warnings

1. Do not dismantle or attempt to remove any components other than those required for routine operations. Any tampering of the device may cause the unit to malfunction and will automatically void the warranty.
2. US FDA restricts the use of this device to sale by or on order of a physician (or properly licensed practitioner).
3. Device is intended to assist in monitoring positive-airway pressure devices' pressure characteristics and should not be relied on as a fail-safe for any connected devices. Device should be monitored continuously by persons having adequate training. Do not leave patients unattended.
4. Do not clean or perform maintenance while device is in use.
5. Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
6. Use of any accessories other than those specified by VORTRAN could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
7. Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 in) to any part of this device, including cables specified by VORTRAN. Otherwise, degradation of the performance of this equipment could result.

VORTRAN APM-Plus Operating Instructions (continued)

On the first Measurement Display screen, the LCD will show PIP, PEEP, and RATE.

PIP: XX cm-H ₂ O
PEEP: XX cm-H ₂ O
RATE: XX BPM
50 2 35 10

PIP from 0 to 55 cm-H₂O, resolution of 1 cm-H₂O
PEEP from 0 to 55 cm-H₂O, resolution of 1 cm-H₂O
RATE from 0 to 99 BPM, resolution of 1 BPM

On the second Measurement Display screen, the LCD will show I-TIME, E-TIME, and IE RATIO.

I-TIME: X.X s
E-TIME: X.X s
IE RATIO: 1:X.X
50 2 35 10

I-TIME from 0 to 9.9 s, resolution of 0.1 s
E-TIME from 0 to 9.9 s, resolution of 0.1 s
IE RATIO from 1:0.0 to 1:9.9, resolution of 0.1

- b. **Alarms:** The VORTRAN APM-Plus monitors the connected cycled-pressure device and alerts clinicians of any abnormality. The current/default Alarm Settings are shown on the last line of the LCD screen. The Alarm Settings can be changed at any time by entering the Alarm Settings menu (see Section 2c – 2d of this guide). There are five high priority alarm conditions in Resuscitator mode (see below). During each condition, the alarm condition will be shown on the LCD and the red LED will flash at 1.4 Hz, with an audible alarm. The current alarm condition that was triggered will be shown on the LCD screen until the condition is corrected. With the exception of the Low Battery Alarm, the operator should correct the alarm condition by consulting the manufacturer of the connected positive airway-pressure device for instructions on stabilizing the performance of their device in order to return the pressure characteristics to the desired range.

PIP: XX cm-H ₂ O
PEEP: XX cm-H ₂ O
HIGH PIP
50 2 35 10

HIGH PIP Alarm - occurs when PIP meets or exceeds the High PIP Setting (shown circled)

PIP: XX cm-H ₂ O
PEEP: XX cm-H ₂ O
LOW PIP
50 2 35 10

LOW PEEP Alarm - occurs when PEEP meets or falls below the Low PEEP Setting (shown circled)

PIP: XX cm-H ₂ O
HIGH RATE
RATE: XX BPM
50 2 35 10

HIGH RATE Alarms - occurs when cycling rate meets or exceeds the High Rate Setting (shown circled)

PIP: XX cm-H ₂ O
PEEP: XX cm-H ₂ O
NON-CYCLING
50 2 35 10

NON-CYCLING Alarm - occurs when connected device stops cycling for a duration greater than or equal to the Non-Cycling Setting (shown circled)

PIP: XX cm-H ₂ O
PEEP: XX cm-H ₂ O
LOW BATT
50 2 35 10

LOW BATT Alarm - occurs when device detects that batteries need replacing.


VORTRAN APM-Plus Operating Instructions (continued)

4. PAP Mode Operation

- a. **Auto-Set:** In PAP Mode only, the Nominal Pressure Setting can be automatically set to the current Real-Time Pressure measurement for the Drift Alert (see Section 4d) by pressing and holding the Auto-Set Button (3) while in the Measurement Display screen. An alert tone will be audible when the change is successful, and the Nominal Pressure Setting shown on the last line in the Measurement Display screen will be updated.

Note: If the Auto-Set feature is used within the first two minutes of operation, the initial two minute mute function will be cancelled (see Section 2e) and the audible alarm can resume its normal operation.

- b. **Monitoring:** The Measurement Display screen in PAP Mode consists of a single screen showing the current Real-Time Pressure measurement.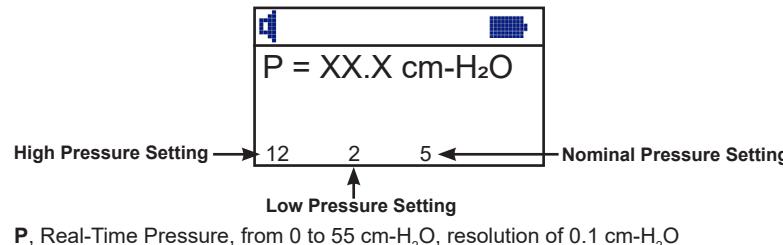


- c. **Alarms:** The VORTRAN APM-Plus monitors the connected constant-pressure device and alerts clinicians of any abnormality. The current/default Alarm Settings are shown on the last line of the LCD screen. The Alarm Settings can be changed at any time by entering the Alarm Settings menu (see Section 2c – 2d of this guide). There are three high priority alarm conditions in PAP mode (see below). During each condition, the APM-Plus provides an audible alarm and a red LED flashing at 1.4 Hz. The current alarm condition that was triggered will be shown on the LCD screen until the condition is corrected. With the exception of the Low Battery Alarm, the operator should correct the alarm condition by consulting the manufacturer of the connected positive airway-pressure device for instructions on stabilizing the performance of their device in order to return the pressure characteristics to the desired range.


P = XX.X cm-H ₂ O HIGH PRESSURE 12 2 5 HIGH PRESSURE Alarm - occurs when Real-Time Pressure meets or exceeds the High Pressure Setting.	P = XX.X cm-H ₂ O LOW PRESSURE 12 2 5 LOW PRESSURE Alarm - occurs when Real-Time Pressure meets or falls below the Low Pressure Setting.	P = XX.X cm-H ₂ O LOW BATT 12 2 5 LOW BATT Alarm- occurs when device detects that batteries need replacing.

VORTRAN APM-Plus Operating Instructions (continued)

- d. **Drift Alert:** In PAP Mode only, if the Real-Time Pressure deviates from the current Nominal Pressure Setting by $\geq \pm 2$ cm-H₂O, an alert tone will be audible and an indication will be shown on the LCD every ten seconds to let the clinician know that the pressure has "drifted."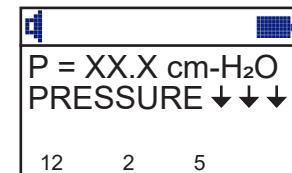


PRESSURE DROP Alert - occurs when Real-Time Pressure falls to 2 cm-H₂O less than the Nominal Pressure Setting.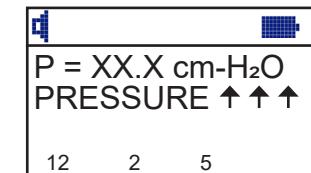

PRESSURE RISE Alert - occurs when Real-Time Pressure rises to 2 cm-H₂O over the Nominal Pressure Setting.

Maintenance and Cleaning

- Changing Batteries** - The VORTRAN APM-Plus operates on two 1.5V AA batteries. When the LCD flashes "LOW BATT," it is time to replace the batteries.
 - Remove VORTRAN APM-Plus from any connected devices.
 - Remove battery cover.
 - Remove old batteries from the compartment.
 - Replace with two new 1.5V AA batteries of the same type (IEC type LR6 or ANSI type 15A).
 - Replace battery cover.
 - Refer to battery manufacturer's specifications for battery shelf-life
- Cleaning and Disinfecting the APM-Plus for Reuse** - Clean between use on each patient or monthly if used continuously on a single patient
 - After use, disconnect the VORTRAN APM-Plus.
 - Clean all exposed surfaces of the VORTRAN APM-Plus by wiping with mild detergent (a suitable detergent is Alconox Powdered Precision Cleaner), 70% isopropyl alcohol, or by spraying with an alcohol based sanitizer (a suitable sanitizer is ALPET D2 available from Best Sanitizers Inc., P.O. Box 1360, Penn Valley, CA 95946, www.bestsanitizers.com).
 - Allow to dry completely before reusing.

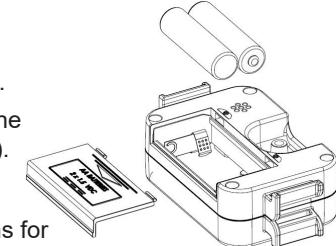NOTE: Do not submerge the VORTRAN APM-Plus in cleaning solution. Do not autoclave.

NOTE: Accessories are single-use and not intended for reprocessing, cleaning, disinfection, or sterilization.

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